



JUST FLOW F Series flowmeters can be used in medical facilities with an available gas piped network.

- Available for oxygen and medical air (*consult our staff for other gases*).
- Scales of 4, 6 or 15 l/min, twistable 360° to facilitate its reading.
- Available with a humidifier bottle, selector-bearing direct outlet. By turning a small side handle or a revolver system, the selector diverts the flow of the humidifier to a tube spigot.
- It can be provided individually or on a dual version, and it can be mounted either with a direct connector, a clamp to a rail or a Ø38.0 mm bar flange. It is fed by a flexible hose with different European and American connectors (*see Connectors and Outlets*).

TECHNICAL SPECIFICATIONS

- Manufactured in anodized aluminium, polycarbonate, polyamide and chromed-plated brass.
- **Outlet connection:** DISS 9/16".
- **Supply:** 4,5 bar.
- **Calibration:** 101,3 kPa and 23°C.
- **Precision:** ± 10 %.
- **Dimensions:**
 - Simple model: Ø31 x 140 mm
 - Double output handle selector: Ø35 x 151 mm.
 - Double output revolver selector: Ø31 x 160 mm
- **Weight:**
 - Simple model: 130 gr.
 - Double output handle selector: 180 g.
 - Double output revolver selector: 215 g.
- In compliance with UNE-EN ISO 15002 standard.
- GNMD Code 61365

Flow range 0 a 15,0 l/min references

	DIN 13260	NF S90-116	BS 5682
Just Flow F + Connector	11FLSXDN15	11FLSXNF15	11FLSXBS15
Just Flow F Double/O + Connector	11FLUXDN15	11FLUXNF15	11FLUXBS15
Just Flow F Double mounting	11FRSXDN15	11FRSXNF15	11FRSXBS15
Just Flow F Double mounting - Double/O	11FRUXDN15	11FRUXNF15	11FRUXBS15
Just Flow F Rail - Knob clamp	13FPSXDN15	13FPSXNF15	13FPSXBS15
Just Flow F Rail - Pushbutton clamp	13FLSXDN15	13FLSXNF15	13FLSXBS15
Just Flow F Rail - Double/O - Knob clamp	13FPUXDN15	13FPUXNF15	13FPUXBS15
Just Flow F Rail - Double/O - Pushbutton clamp	13FLUXDN15	13FLUXNF15	13FLUXBS15
Just Flow F Rail - Double/M - Knob clamp	13FTSXDN15	13FTSXNF15	13FTSXBS15
Just Flow F Rail - Double/M - Pushbutton clamp	13FSSXDN15	13FSSXNF15	13FSSXBS15
Just Flow F Rail - Double/M-Double/O-Knob cl.	13FTUXDN15	13FTUXNF15	13FTUXBS15
Just Flow F Rail - Double/M-Double/O-Pushb. cl.	13FSUXDN15	13FSUXNF15	13FSUXBS15
Just Flow F Ø38,0 mm flange	14FLSXDN15	14FLSXNF15	14FLSXBS15
Just Flow F Double/O to Ø38,0 mm flange	14FLUXDN15	14FLUXNF15	14FLUXBS15
Just Flow F Double/M to Ø38,0 mm flange	14FRSXDN15	14FRSXNF15	14FRSXBS15

¹ Medical Air models, replace X with M (please consult our staff about other gases).

² for other flow rates, replace 15 with 04 or 06, depending on flow rates of 4 l/min or 6 l/min (please consult our staff about other flow rates).

³ Please consult our staff about other gas standars.

⁴ Please consult our staff about other models.